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7. Temeljna literatura:

- Autoclaves recommendations, Raypa. (2024). *Essential tips for proper autoclave loading*. [Essential Tips for Proper Autoclave Loading | RAYPA](#)
- Commission implementing decision (EU) 2023/1411. (July 2023). *Amending Implementing Decision (EU) 2021/1195 as regards a harmonised standard for sterilization of health care products*. [Commission Implementing Decision \(EU\) 2023/... of 4 July 2023 amending Implementing Decision \(EU\) 2021/1195 as regards a harmonised standard for sterilization of health care products \(europa.eu\)](#)
- Environmental Health and Safety. (2024). *Autoclaving Guidelines*. [Autoclaving Guidelines | Environmental Health and Safety \(uiowa.edu\)](#)
- ISO 17665:2024.(2024). Sterilization of health care products. *Moist heat — Requirements for the development, validation and routine control of a sterilization process for medical devices*. [ISO 17665:2024 - Sterilization of health care products — Moist heat — Requirements for the development, validation and routine control of a sterilization process for medical devices](#)
- ISO 14937:2009. (2009). Sterilization of health care products. *General requirements for characterization of a sterilizing agent and the development, validation and routine control of a sterilization process for medical devices*. [ISO 14937:2009\(en\), Sterilization of health care products — General requirements for characterization of a sterilizing agent and the development, validation and routine control of a sterilization process for medical devices](#)
- ISO 11138-1:2017. (2017, 2022). Sterilization of health care products. *Biological indicators — Part 1: General requirements*. *International Organization for Standardization*. [ISO-11138-1-2017.pdf \(iteh.ai\)](#)
- ISO 25424:2018. (2018). Sterilization of health care products. *Low temperature steam and formaldehyde — Requirements for development, validation and routine control of a sterilization process for medical devices*. [ISO 25424:2018 - Sterilization of health care products — Low temperature steam and formaldehyde — Requirements for development, validation and routine control of a sterilization process for medical devices](#)
- Hoxey, E. (2019). Medical Device White Paper Series. *Sterilization – Regulatory requirements and supporting standards*. [Sterilization Revised.pdf \(medical-device-regulation.eu\)](#)
- Mecher, S. (2024). Autoclave Standard Operating Procedures. How to Safely Operate an Autoclave. *Preparation, Packaging & Loading of Materials*. [Standard Autoclave Procedures \(SOPs\) Guide + Template \(consteril.com\)](#)
- Piletič, N., Pristavec, T., Kadunc, T., & Pervinšek, S. (2024). Priporočila za delo v sterilizaciji. (Verzija 2.). Zbornica zdravstvene in babiške nege Slovenije – Zveza strokovnih društev medicinskih sester, babic in zdravstvenih tehnikov Slovenije. Sekcija medicinskih sester in zdravstvenih tehnikov v sterilizaciji.
- World Health Organization & Pan American Health Organization (2016). *Decontamination and reprocessing of medical devices for health-care facilities*. [1-118 BenedettaFinal5 \(who.int\)](#)

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